

# Bradys... Providing perfect cover - everytime.

## Dental UltraFine Plaster



- TRADITIONAL formula
- EASIER to apply
- CONSISTENT setting time
- PROVEN reliability
- RECOMMENDED by professional craftsmen
- IDEAL for moulds, casts and dental work

**Cover it with Bradys!**

[bradys.com.au](http://bradys.com.au)

**BRADYS**  
PLASTER PRODUCTS

# Bradys... Building on over 70 years experience!

With a reputation as Western Australia's leading supplier of quality plaster compounds for over 70 years, it comes as no surprise that Bradys plaster products and compounds have been specially formulated for the professional plasterer who is looking for a lightweight and superior product.

Extremely easy to mix, work and apply, Bradys plaster products and compounds provide an excellent bond with either traditional clay bricks or aerated concrete blocks.

We have reduced the risk of product tampering and wastage which means that Bradys plaster products and compounds will also significantly improve on site cleanliness, reduce time delays and lessen demanding labour requirements.

## Bradys Plaster Products and Compounds

Bradys plaster products and compounds are suitable for all residential and commercial requirements.

Bradys proven manufacturing techniques along with world class plant, equipment and facilities ensures that all plasters and compounds meet the stringent demands of regulatory bodies throughout Australia.

The company's pursuance of excellence also ensures Bradys plaster products and compounds are of a premium quality.

## DENTAL ULTRAFINE PLASTER

### Specifications

#### APPLICATION:

1. This plaster has been especially formulated for fine work and is ideal for making moulds and orthopaedic castings that require a very fine, hard and smooth finish.
2. It is ideal for making moulds that require a very fine, hard smooth finish.
3. The setting time of Bradys Dental UltraFine Plaster is approximately 10 minutes.

**NOTE:** Application of Bradys Dental UltraFine Plaster over substrates that contain excess levels of fluidising agents will cause variations and inconsistency in setting time.

#### Mixing:

1. Into a clean mixing bucket, add plaster to clean water at an approximate ratio of 2 volumes plaster to 1 water.
2. Allow Bradys Dental UltraFine Plaster to soak for approximately 2-3 minutes.
3. Mix thoroughly for approximately 15 seconds, or until a smooth even gauge is achieved free from unmixed plaster.

**NOTE:** Do not over mix as over mixing may reduce the setting time of the plaster.

## REMEMBER

STORE IN A COOL DRY PLACE AND NEVER PLACE DIRECTLY ON A CONCRETE FLOOR OR WET SAND.

**BRADYS: HELPLINE - (08) 9374 2900**


#### PRECAUTIONS

- RISK:** The dust and paste may be irritating to the skin, eyes and respiratory system.
- SAFETY:** Avoid contact with eyes and skin. Avoid breathing dust. Wear suitable protective clothing. Wash thoroughly after handling. Wash work clothes regularly. Work in a well ventilated area.
- FIRST AID:** If affected by dust, remove to fresh air. Irrigate eyes with plenty of water for 10 minutes. Wash skin thoroughly with mild soap and water.
- DISPOSAL/SPILLS:** Clean up by wet sweeping or vacuuming and follow above safety precautions. Collect in covered containers for disposal in accordance with local authority guidelines.
- FIRE:** Bradys Dental UltraFine Plaster is a non-flammable product. However, can emit oxides of sulphur and carbon dioxide if heated to decomposition.

Made in WA



#### LOCAL STOCKIST

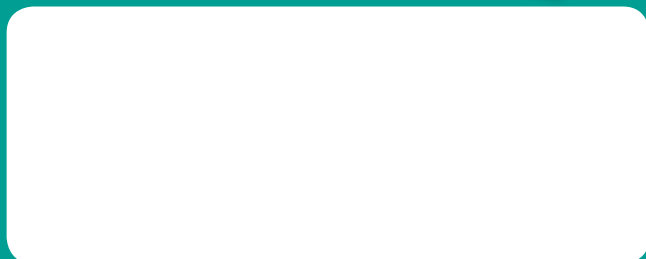

#### Head Office Perth WA

290 Bushmead Road, Hazelmere, Western Australia 6055  
Telephone: (08) 9374 2900 Facsimile: (08) 9374 2901  
Postal Address: PO Box 1994, Midland DC, Western Australia 6936

#### Warranty:

Bradys warrants its products to be free from defects caused by faulty manufacture or materials. If any of its products are so defective the Company will at its option, repair or replace them, supply equivalent replacement or reimburse price. This warranty shall not apply to any loss or consequential loss suffered through or resulting from defects caused by faulty manufacture or materials.

Cover it with Bradys!

[bradys.com.au](http://bradys.com.au)

**BRADYS**  
PLASTER PRODUCTS